

Radiology Solutions Services

DX-D 300

Document ID: 78178328

Overview

This checklist is attached to the DX-D 300 Service Manual.



DX-D 300 Serial Number:	
DX-D 300 installation date:	2021-07-26
Date of maintenance:	2024-11-27
Maintenance interval:	12 months or another interval: ...
Service Engineer name:	ANATOLII KOLYADA

Purpose of this document:

This document provides the reference guide and checklist for the different Preventive Maintenance Visit (PMV) tasks for the complete DX-D 300 System.



IMPORTANT:

Strictly observe the safety advices in the referenced documents.

Perform the PMV tasks as described in the referenced documents. This checklist does only contain a short description of the tasks.

Defective or worn out parts or misaligned components, which are found during the PMV are *not* included in the below stipulated labor time. Parts and labor cost for such replacements or alignments (labor & parts) need to be charged outside of a PMV.

Required time



REQUIRED TIME:

- 12-month Maintenance: Approximately 5h
- Additional activities for IEC test after 3 years: Approximately up to 1h additionally. IEC test depending on local requirements.
Or shorter period for IEC test if required.
- Additional activities after 5 years: For steel cable replacement 1 person, 2h.
Refer to the DX-D 300 - U-Arm - Service Manual with SHF generator, Document ID 41588092,
or
Refer to the DX-D 300 - U-Arm - Service Manual with SHFR generator, ID 81611653

Changes compared to previous revision

Added information about the Sedecal SHFR generator.

DOCUMENT CONTROL NOTE:

The controlled version of this document is available from the Agfa Medimg Library. Any printed copy of this document is uncontrolled.

bx N01/02-04-11
cig 03.02.25

bx N01/02-06-25
cig 05.02.25

Maintenance Checklist

DX-D 300

After each Maintenance

9 Completion of the Preventive Maintenance

9 Completion of the Preventive Maintenance				
Topic	Task	N.A.	OK / Done	
Results	Explain results to customer.	<input type="checkbox"/>	<input checked="" type="checkbox"/>	#
Checklist	<ul style="list-style-type: none">Archive the maintenance work according to your local regulations. For example attach the completed checklist to ASSM PMV Workorder.Hand out this checklist to the customer as needed.		<input checked="" type="checkbox"/>	#

Comments for Completion of Preventive Maintenance

#	Description
1	
2	



* If applicable, fill in a comment number and add the comment description in the comment table.

DOCUMENT CONTROL NOTE:

The controlled version of this document is available from the Agfa Medimg Library. Any printed copy of this document is uncontrolled.